audifon GmbH & Co. KG
Werner-von-Siemens-Straße 2
99625 Kölleda
Germany

Phone: +49-3635-4056-590
Fax: +49-3635-4056-589

contact@audifon.com

www.audifon.com

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OVERVIEW OF THE audifon charge & dry

This manual guides you through the use of your new audifon charge & dry. Please read these instructions carefully before using the equipment. If you have any further questions, please contact your hearing aid specialist.

Intended use:

The power case is a class I medical device designed to charge and maintain your rechargeable hearing aids audifon lewi R Li & risa R Li, mobile mode or other sources > 500 mA. The power base is a class I medical device which consists in a compatible station with the power case. The power base is designed to provide the additional drying function to the power case.

Intended user / User group:

People with hearing loss who use hearing aids and their carers • Hearing aid specialist responsible for the user • Adults and children over 16 years old.

Usage environment:


Refer to section "Conditions of use, transport and storage" page 42.

COMPONENTS OF THE audifon charge & dry


Product description

power case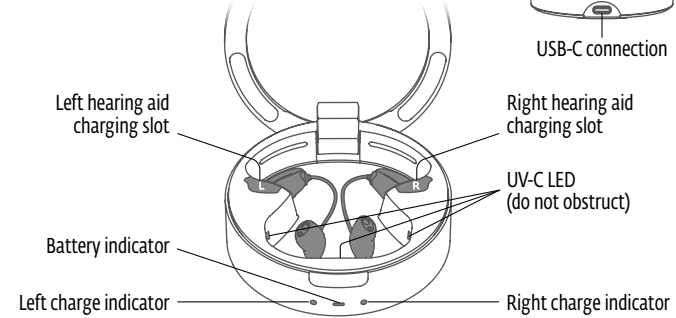

power case and power base pairing

power base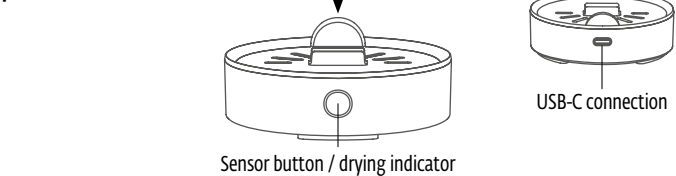

HOW TO USE THE power case

You can use the power case in three different ways.

① Use on electrical socket or USB (output ≥ 500 mA)


When the power case is plugged in, the following functions are available: charging the audifon lewi R Li & risa R Li, charging the internal battery, UV-C treatment.

② Mobile use

If no power outlet is available, the power case can be operated by its internal battery. Only the audifon lewi R Li & risa R Li charging function is available.

③ Use on the power base

When the power case is positioned on the power base (plugged in), the following functions are available: charging the audifon lewi R Li & risa R Li, charging the internal battery, UV-C treatment and drying the audifon lewi R Li & risa R Li.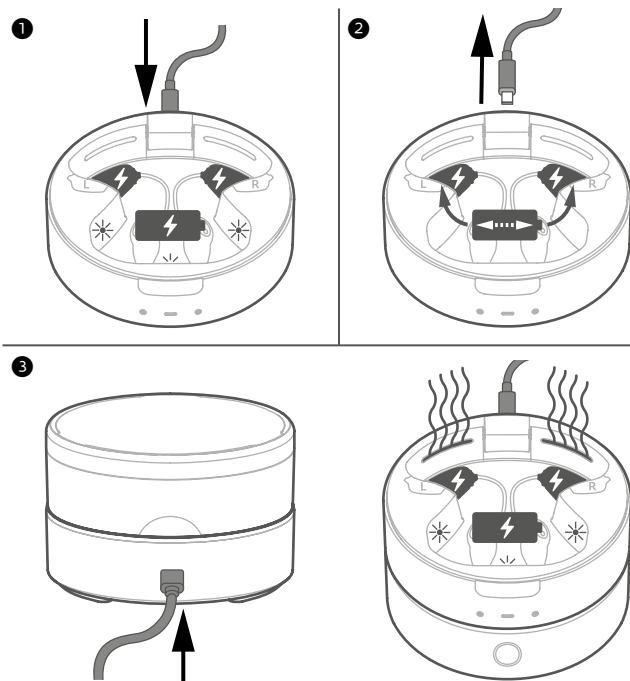

CONNECTION OF THE power case AND THE power base

Preparing the power case for use with an electrical socket or USB:


- ❶ Insert the USB cable into the back of the power case.
- ❷ Insert the plug into a wall socket or USB (output ≥ 500 mA).
- ❸ When the power case is connected, the indicator lights glow blueish white for 5 seconds.

Preparing the power base for use with the power case:


- ❶ Insert the USB cable into the back of the power base.
- ❷ Insert the plug into a wall socket only.
- ❸ When the power case is connected to power base, all indicator lights glow blueish white for 5 seconds.

Note: All functionalities are working by using a power plug with at least 2A.

power case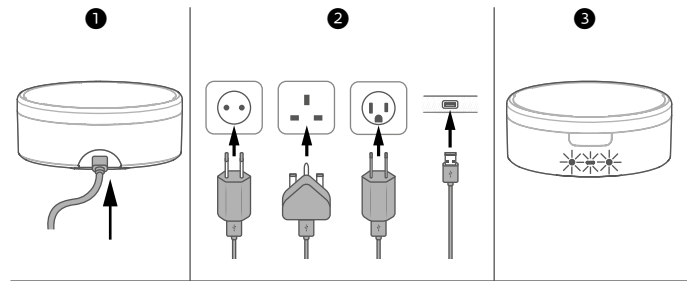

power base + power case

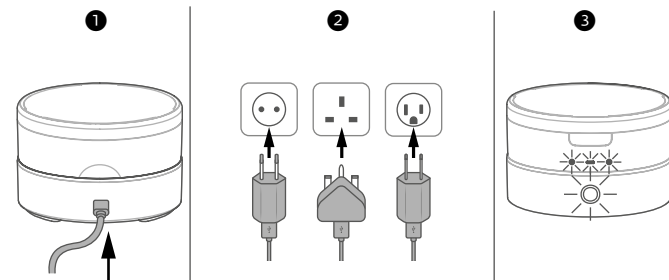CHARGING AND UV-C TREATMENT OF
audifon lewi R Li & risa R Li

- 1 Insert the audifon lewi R Li & risa R Li in the designed charging slots of the power case (simultaneously or independently), the charge starts and indicators blink for 20 seconds. audifon lewi R Li & risa R Li are fully charged in about 2.5 hours.
- 2 The UV-C treatment cycle works while the audifon lewi R Li & risa R Li are charging. It starts once the lid is closed and lasts 1 hour.

Note: The UV-C treatment cycle starts only once the power case is connected to the electrical and USB outlets (output ≥ 500 mA) or when placed on the power base. The charge of the audifon lewi R Li & risa R Li and UV-C treatment cycle happen simultaneously but independently. Opening the lid does not interrupt the audifon lewi R Li & risa R Li charging but will pause the UV-C cycle. If the lid is not closed after 10 seconds, the UV-C cycle treatment will automatically stop. Once closed again, an entirely new UV-C treatment cycle will start.

Note: If one audifon lewi R Li & risa R Li is removed during the charge or UV-C cycle, the cycles will not be impacted (they both continue). If one audifon lewi R Li & risa R Li is added during the cycle, the charging cycle will not be impacted, however, the UV-C cycle will reset.
- 3 The charging process stops automatically when the audifon lewi R Li & risa R Li batteries are fully charged. They can be left in the power case. The UV-C treatment process automatically stops after 1 hour. Turn the audifon lewi R Li & risa R Li back on before use.

Note: Remove the audifon lewi R Li & risa R Li from the power case without pulling on their receiver unit as this may damage them.

audifon lewi R Li & risa R Li charge status indicator

To view the status of the audifon lewi R Li & risa R Li during the charging cycle, open the lid of the power case and the status will be displayed for 20 seconds.

LED	Signification
 Flashing orange	 Empty < 5%
 Constant orange	 Normal 5% - 80%
 Constant green	 Full 80% - 100%

CHARGING THE INTERNAL BATTERY OF THE power case

You can recharge the internal battery of the power case in two different ways.



- 1 From an electrical socket or USB (output ≥ 500 mA).
See section "Connection of the power case and the power base", page 28.
- 2 With the power base
See section "Connection of the power case and the power base", page 28.

Full charging of the internal battery will take approximately 4 hours. The charging process stops automatically when the battery is fully charged.
The internal battery of the power case is charged simultaneously with the charging, UV-C treatment and drying of the audifon lewi R Li & risa R Li.

Note: Opening the lid does not interrupt the charge of the internal battery.
Make sure that the internal battery is sufficiently charged before using it in mobile mode. A fully charged internal battery enables approximately 3 complete charging cycle of the audifon lewi R Li & risa R Li.

power case internal battery status indicator

To view the status of the internal battery during the cycle, open the power case lid and the status will be displayed for 20 seconds.

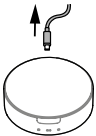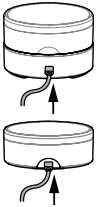Supply	Action	LED	Signification
 Without power supply	Open the lid / right after supply disconnection	 Constant green	Multiples recharges of the hearing aid pair
		 Constant orange	1 remaining recharge of the hearing aid pair
		 Flashing orange	Battery empty
 With power supply	Open the lid	 Contant green	Charging cycle is finished
		 Constant orange	Charging cycle is in progress
	Close the lid / at supply connection	 Glowing blueish white 15 s.	The device is correctly powered

DRYING THE audifon lewi R Li & risa R Li (ONLY WITH power base)

See section "Connection of the power case and the power base" on page 28 for installation of the power base.

You can pair the power case and the power base in two different ways:

Option 1

- ➊ Insert the audifon lewi R Li & risa R Li into the intended ports in the power case, then close the lid.
- ➋ Place the power case on the power base. The pairing of the power base and power case is displayed for 5 seconds with a glowing blueish white pattern on all indicator lights of both products.
- ➌ The charging of the audifon lewi R Li & risa R Li and the internal battery, the UV-C treatment cycle and the drying cycle start as soon as the pairing is done between the power case and the power base.
- ➍ The drying cycle takes 60 minutes.
- ➎ The drying cycle stops automatically. Remove the audifon lewi R Li & risa R Li from the charging slots in the power case. audifon lewi R Li & risa R Li can also be kept in the power case after the cycle.

Option 2


- ➊ Place the power case on the power base. The pairing of the power base and power case is displayed for 5 seconds with a glowing blueish white pattern on all indicator lights of both products.

- ➋ Insert the audifon lewi R Li & risa R Li into the intended charging slots on the power case, then close the lid.
- ➌ To start the drying cycle, press the sensor button on the power base. The drying cycle takes 60 minutes.
- ➍ The drying cycle stops automatically. Remove the audifon lewi R Li & risa R Li from the charging slots in the power case. audifon lewi R Li & risa R Li can also be kept in the power case after the cycle.

General information:

- The drying cycle can be paused by touching the sensor button. A second press restarts the cycle.
- Opening the lid stops the drying cycle. To restart, the lid has to be closed and the central button should be pressed.
- The audifon lewi R Li & risa R Li charge, the internal battery charge, the UV-C treatment cycle and the drying cycle are simultaneous but independent.
- After 60 minutes of drying, the audifon lewi R Li & risa R Li are optimally dried. If the audifon lewi R Li & risa R Li are still charging at the end of this period, drying cycle is extended to hold the temperature (temperature holding phase).

power base drying status indicator

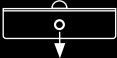	Signification
 Glowing blueish white for 15 seconds	audifon lewi R Li & risa R Li are present, drying cycle starts or resumes
 Flashing red for 10 seconds	No power case detected or pairing error and button pressed
 Flashing orange for 5 seconds	Drying cycle is paused or no audifon lewi R Li & risa R Li is detected or lid is open

MAINTENANCE OF THE audifon charge & dry

Before use:

Before use, make sure, that the audifon lewi R Li & risa R Li and the audifon charge & dry are dry and clean.
Check that the charging slots for the audifon lewi R Li & risa R Li and the UV-C LEDs are clean.
No material should obstruct the reception of audifon lewi R Li & risa R Li and the UV-C LEDs.

Cleaning:

If cleaning is required, disconnect the charger from the power supply and ensure that the audifon lewi R Li & risa R Li are removed from the power case.
Clean the charger with a soft, dry cloth.
Do not use water or cleaning fluid to clean the charger.

TROUBLESHOOTING

power base	
	
Drying indicator LED in red:	
Check if a power case is correctly connected on the power base	
If a power case is correctly connected on the power base and drying indicator LED is still red, disconnect the USB-C cable, wait 30 seconds, connect again the USB-C cable	
If drying indicator LED is still red, please contact your hearing aid specialist	
power case	
	
Left charging LED in red:	Right charging LED in red:
Remove left/right audifon lewi R Li & risa R Li from its charging slot, wait 30 seconds	
Close the power case lid	
If left charging LED is off, open the lid and put again the left audifon lewi R Li & risa R Li in its charging slot	If right charging LED is off, open the lid and put again the right audifon lewi R Li & risa R Li in its charging slot
If left charging LED is still red, please contact your hearing aid specialist	If right charging LED is still red, please contact your hearing aid specialist

Left and right charging LEDs in red (in the same time):
Remove left and right audifon lewi R Li & risa R Li from their charging slots, wait 30 seconds
Close the power case lid
If left and right charging LEDs are off, open the lid and put again the left and right audifon lewi R Li & risa R Li in their charging slots
If left and right charging LEDs are still red, disconnect the USB-C cable, wait 30 seconds, connect again the USB-C cable
If left and right charging LEDs are still red, please contact your hearing aid specialist

Battery indicator LED in red:
If the power case is used alone : Disconnect the USB-C cable, wait 30 seconds, connect again the USB-C cable
If the power case is used with a power base : Disconnect the power case from the power base, wait 30 seconds, connect again the power case on the power base
If battery indicator is still red, please contact your hearing aid specialist

IMPORTANT SAFETY INFORMATION

- Choking hazard. Keep out of reach of children under 3 years, cognitively impaired persons and pets.
- Do not cover the power case or the power base during operation.
- Only put power case or power base on stable surface. Do not place power base or power case on carpet or any other element that could obstruct the air inlets.
- Risk of interference. For people with active implantable medical devices, it is recommended that a distance of at least 15 cm be maintained between the power case or power base and the implant.
- Risk of electric shock. Use only the original adapter and cable. Do not use equipment other than that supplied by the manufacturer.
- Risk of battery damage. Never expose the power case or power base to temperatures above 45 °C or to prolonged exposure to sunlight, open flames or other sources of inflammation.
- Never attempt to repair the power case or power base yourself. Contact your hearing aid specialist if you notice any anomalies.
- Risk of product damage. Never place objects other than your hearing aids in the charging slots provided in the power case.
- If, despite the safety system, the ultraviolet (UV) light remains on with the charger lid open, do not look directly at the UV light and contact your hearing aid specialist.
- Risk of injury. If the power case's internal battery leaks, do not carry it as this may cause skin irritation. If you come into contact with the battery fluid, use a dry cloth to wipe it off. If you experience any skin irritation, consult your doctor.
- Do not use the power case or power base in explosive atmospheres.
- Do not use the power case or power base, if it is obviously damaged.
- As a precaution, please disconnect the device in the event of a thunderstorm.

If you experience problems, contact a hearing aid specialist. All serious incidents related to the device should be the subject of a notification to the manufacturer and the competent authorities in the member state in which the user is located.

BATTERY PRECAUTIONS

The device is equipped with a battery (Lithium-Ion).

Do not replace with an incorrect type.

Do not short-circuit, risk of explosion.

Do not dispose of in fire or by crushing, risk of explosion.

Use only the reference battery supplied with the product.

Unauthorized replacement or modification of the battery may damage the device or cause injury.

CONDITIONS OF USE, TRANSPORT AND STORAGE

Operating conditions

Temperature: 10 °C - 35 °C

Humidity: 20 % - 75 %

Transport and storage conditions

Temperature: -20 °C - 60 °C

Humidity: 5 % - 90 %

Atmospheric pressure: 750 hPa - 1060 hPa

DISPOSAL INFORMATION

Dispose of rechargeable product according to local regulations. Please recycle when possible.

Do not dispose the rechargeable product as household waste or attempt to burn it as it may explode. Batteries are harmful to the environment. Dispose of your used hearing aid charger according to your country's regulations or return it to your hearing care professional.

TECHNICAL CHARACTERISTICS

power case

External dimensions: Ø 83 x 32 mm

Weight: 109 g

UV LED wavelength: 260 nm - 280 nm

UV cycle time: 60 min

Hearing aid charge cycle time: ~ 2 h 30

Internal battery charge cycle time: ~ 4 h

power base

External dimensions: Ø 83 x 32 mm

Weight: 87 g

Drying temperature: 35 °C

Drying cycle time: 60 min

Technical characteristics of the power supply

Input voltage: 100 V - 240 V

Input current: 0.3 A

Input frequency: 50 - 60 Hz

Output voltage: 5 V

Output current: 2 A

INFORMATION ON COMPLIANCE

Regulation applicable to medical devices

This product meets the requirements of Regulation (EU) 2017/745 covering medical devices.

GUARANTEE

24 months - (USB cable and power supply included). In the event of a malfunction, contact your hearing aid specialist. Proof of purchase will be requested for all work by the customer service during the guarantee period. This guarantee does not apply to defects caused by an accident or incident or similar damage, by introducing a liquid, negligence, incorrect use, dismantling of any kind, lack of care or any other circumstance caused by the user. Furthermore, this guarantee will not apply to damage caused by lightning or any other electric voltage variation, or unauthorized modifications or replacement.

SYMBOLS

 This product is compliant with Regulation (EU) 2017/745 on medical devices.	
 Manufacturer	 Date of manufacture
 Distributor	 Medical device
 Serial number	 Unique Device Identifier
 Catalogue reference	5V-0.4A  Direct current
 Batch number	 Indoor use only
 UK Conformity Assessed	 Regulatory compliance mark
 The product is made in ABS with a recycling code of 7	 Consult instructions for use
 Li-Ion battery recycling	 Sorting instructions
 DISPOSAL - Dispose of old appliances ecologically. Old devices contain recyclable materials which must be recovered. Therefore, make sure old devices are disposed of using the appropriate collection systems.	

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

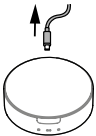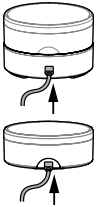Supply	Action	LED	Signification
 Without power supply	Open the lid / right after supply disconnection	 Constant green	Multiples recharges of the hearing aid pair
		 Constant orange	1 remaining recharge of the hearing aid pair
		 Flashing orange	Battery empty
 With power supply	Open the lid	 Constant green	Charging cycle is finished
		 Constant orange	Charging cycle is in progress
	Close the lid / at supply connection	 Glowing blueish white 15 s.	The device is correctly powered

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FCC PART 15 - DECLARATION OF CONFORMITY

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help